

Eye Protection Re-Use & Cleaning Guidelines



Goggles/Face Shield:

- Goggles & Face Shields are re-usable.
- Perform hand hygiene before and after touching eye protection.
- Wash eye protection immediately, if touched with dirty hands.
- Discard at point of use, if visibly soiled with blood, body fluids or hazardous drugs, per hospital policy.

Two (2) Cleaning Methods:

1. Hospital approved disinfectant

Using soft cloth or pre-moistened wipe, disinfect with non-alcohol containing disinfectant.

Examples: Green or Grey Top Sani-Cloth.

2. Soap and water

Wash with mild soap & warm water.

Dry with wash cloth or paper towel.

- Store in a clean, dry manner, in a way that prevents contamination, until next use.

